



**"GattaCo's technology may prove to be
a game-changer for blood testing."**

Dr. Anders Berg, Vice-Chair and Head of CLIA &
Lab Operations, Cedars Sinai Medical Center

**Don't stand still.
Skate to
the puck!**

**Self-collected
Lab Quality Plasma
ANYWHERE**



Seamlessly Fits Your Workflow


Expand Your Market Access



GATTACo™ BENEFITS	Testing Labs	Clinical Studies	Point-of-Use
Anywhere Convenience	Remove lab visits	Increase participant diversity & retention	Access users directly
Lab-Quality Ready to Use	Fewer steps, less sample prep error	Fewer steps, less sample prep error	Provide plasma-ready sample
Easy to Integrate	Streamline workflow	Streamline workflow	Fits existing tests
Cost Effective	Less backend logistics	Eliminate phlebotomist	Eliminate phlebotomist

The A-PON device removes the need for phlebotomy, and easily allows consumers to convert fingerstick blood to liquid plasma in under 3 minutes without any specialized knowledge or equipment. The plasma obtained fits seamlessly into existing testing workflows without changing established thresholds.

Seamless Fits Your Workflow

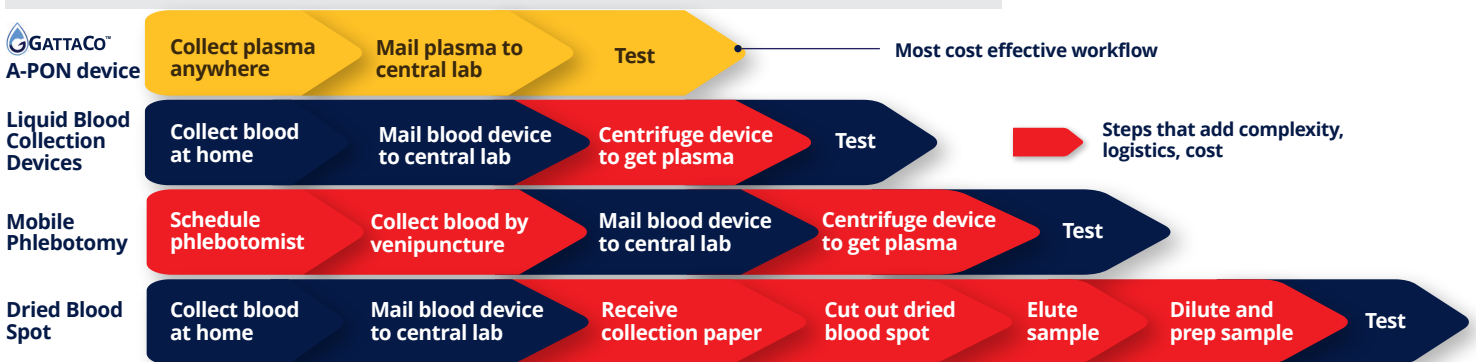


It's easy. Try it now. gattaco.com

See video, data. Contact us:



Covid-19 Neutralizing Antibody Assay (Natural and Spiked Blood Samples)

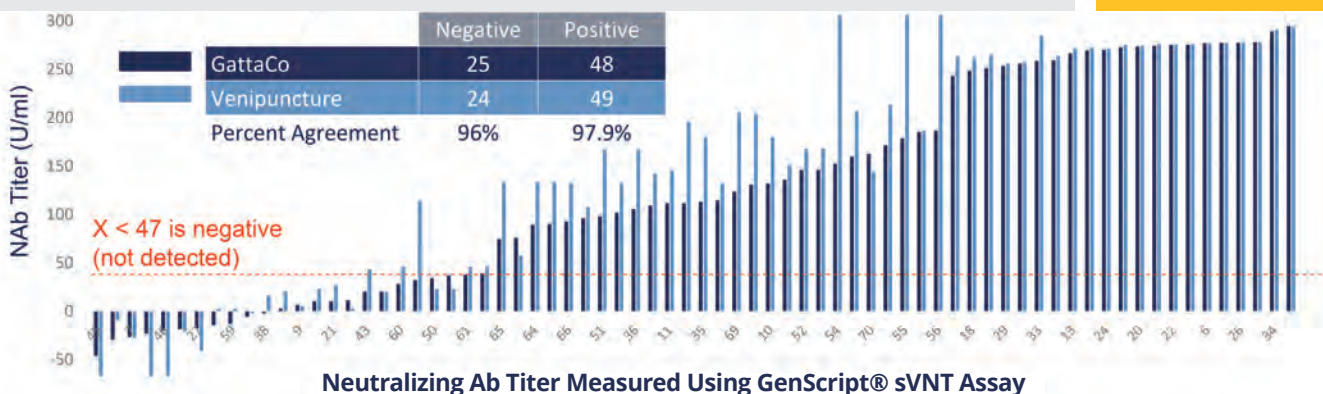

Kidney Disease Panel with Proprietary BioMarkers

STUDY	GLUCOSE			POTASSIUM			CREATININE			CKD-BIOMARKER			GLYCEMIC-BIOMARKER		
	CONTROL	GATTACo	24HR	CONTROL	GATTACo	24HR	CONTROL	GATTACo	24HR	CONTROL	GATTACo	24HR	CONTROL	GATTACo	24HR
1	79.7	77.0	57.3	3.9	4.3	6.9	0.7	0.8	0.7	3.9	4.1	3.8	13.5	13.4	13.4
2	162.7	152.3	144.3	4.0	4.8	6.1	0.9	0.8	0.8	5.4	5.4	5.5	15.5	15.5	15.6
3	83.0	83.0	64.0	3.8	4.5	7.6	0.7	0.7	0.6	4.4	4.1	4.3	12.4	12.4	12.2
4	81.7	80.3	60.0	4.8	5.1	8.1	1.0	0.9	1.0	4.8	4.9	4.9	13.5	13.6	13.7
5	80.0	77.0	56.0	4.0	4.5	7.9	1.1	1.1	1.1	4.0	3.6	3.9	14.0	14.1	14.7

©2022 GattaCo
All Rights Reserved.
A-PON device is for Research Use Only
Not FDA-Cleared

Study performed with CLIA Lab partner. All samples are Li-Heparin plasma.
 • "Control" is plasma separated by centrifuge immediately after blood collection.
 • "GattaCo" is plasma separated immediately after blood collection using the GattaCo device.
 • "24hr" is plasma separated by centrifuge 24 hours after blood collection.